

How to make an E-collar (the templates for everything at full scale.)

Step 1: Trace template onto clear plastic notebook cover, then cut out

Step 2: 1 sheet Dr. Scholl's Molefoam: cut 2 stripes 1/4" along long edge

Step 3: Remove one adhesive strip and stick one on top of onther leaving one adhesive strip for later

Step 4: Cut 3/4" off of dual layer molefoam - remove backing and adhere to clear plastic along one edge

Step 5: 1 Sheet Dr. Scholl's Mole Skin - Cut a stripe 1 3/8" wide down length

Step 6: Cut slits 1/2" deep at 1/2" intervals down length

Step 7: Turn collar over, molefoam side down.

Remove backing from moleskin. Affix uncut edge to collar - About 1/4" of it

Step 8: From the middle of moleskin - wrap around over mole foam, maintaing contours. Trim excess moleskin from edge.

Step 9: Cut 1/2" Strip from moleskin

Step 10: Remove backing and adhere over slitted moleskin.

Step 11: Cut 1 strip 1 3/4" long X 5/8" wide

Step 12: Remove backing and place on edge of collar

Step 13: Wrap around edge of collar

Mole skin Step 9

2x Mole foam Step 2

Mole skin Step 5-6

Mole skin
Step 10

Template (full size)

